

MED<sup>9</sup>EL

# SAMBA 2

Audio Processor




## Quick Guide

# Table of Contents

Changing the Battery . . . . .	2
Changing the Cover . . . . .	4
Using Attachment Clips . . . . .	5
Optional Accessories . . . . .	6
Troubleshooting . . . . .	8
Basic Care and Maintenance . . . . .	9

The SAMBA 2 Quick Guide is intended as an easy-to-use reference when handling your audio processor. For detailed information on the intended use, indications, contraindications, warnings and precautions of SAMBA 2, please refer to the instructions for use.

# SAMBA 2 Audio Processor

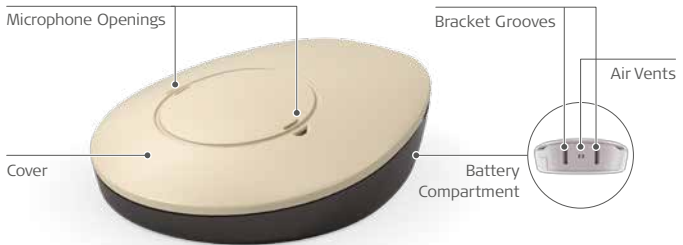

# Changing the Battery




- 1** Pull on the raised lip to swing the battery compartment open sideways.



- 2** Turn the audio processor upside down to let the old battery fall out.



- 3** Remove the protective film from the new battery.

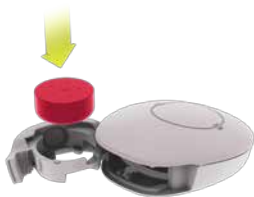

- 4** Place the new battery in the compartment, with the positive (+) pole facing upwards.



- 5** Close the battery compartment.

**!** It is recommended to remove the protective film one to two minutes before battery insertion. This time is needed for the zinc-air battery to charge up completely.

# Changing the Cover




**1** Position your finger under the edge of the cover. Gently lift up to remove the cover.



**2** Position the new cover over the processor and push down until it snaps into place.

# Using Attachment Clips

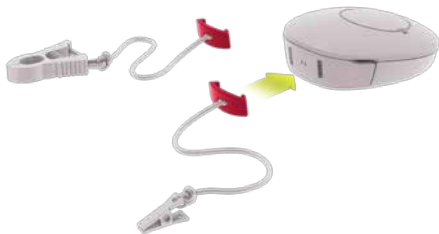

**1** Position the bracket of the attachment clip over the bracket groove.

**2** Push until the bracket snaps into place.

**3** Clip to hair or clothes as appropriate.


## 6 SAMBA 2

# Optional Accessories

Thanks to its advanced hearing technology, you don't need any additional accessories to use your SAMBA 2. However there are two handy accessories available, which you can use for your own convenience.

### SAMBA 2 Remote

The SAMBA 2 Remote is an optional mobile app which can assist with switching between simple audio processor settings, as predefined by your audiologist or healthcare professional. The SAMBA 2 Remote app is available for both iPhone and Android, and can be downloaded separately from your app store.

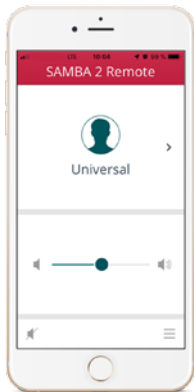



### SAMBA 2 GO

SAMBA 2 GO is an optional assistive listening device which can be used to stream sound from your mobile phone, TV, tablet, and many other devices. It can help with making hands-free phone calls, and brings the sound from your device straight to your ears. Additionally, SAMBA 2 GO can assist with switching between simple audio processor settings, as predefined by your audiologist or healthcare professional. SAMBA 2 GO is available separately.



# Troubleshooting

If you encounter any minor technical problems, carefully follow these steps to identify possible causes and solutions:

1. Check the polarity of the battery. The flat side (+) must be on top.
2. Check if the battery is empty and needs to be replaced.
3. Check the audio processor for dirt or signs of damage. Gently clean any processor parts that are dirty and wipe dry after cleaning. If any parts appear damaged, contact your MED-EL representative.
4. Read detailed information on troubleshooting in your SAMBA 2 instructions for use.
5. Contact your audiologist, healthcare professional, or your MED-EL representative.

# Basic Care and Maintenance

Follow these basic yet important care instructions to help improve the functioning of your SAMBA 2 Audio Processor.

## Cleaning

MED-EL recommends cleaning SAMBA 2 weekly for hygienic reasons. Only clean the outside of the device. Do not clean SAMBA 2 in or under water. Use a damp cloth to gently clean your audio processor. If necessary, use non-abrasive household soap together with a damp cloth. Do not use aggressive cleaning agents.

## Storage

When not in use, SAMBA 2 should be kept in the case provided. To extend the battery life, you can remove the battery from SAMBA 2, or simply open the battery compartment, when not in use.

## Contact with Water

Prevent water from running into SAMBA 2 via the microphone openings or the battery compartment.



### Drying

If you live in a humid climate or perspire heavily, the audio processor should be placed in a drying kit instead of in the SAMBA 2 case when it is not worn.



### Sunlight

Please store the SAMBA 2 in a dry place and protect it from direct sunlight.



Refer to your instructions for use for more information about cleaning, storage, handling and disposal of the SAMBA 2 Audio Processor.









MED-EL Elektromedizinische Geräte GmbH  
Fürstenweg 77a | 6020 Innsbruck, Austria  
office@medel.com

medel.com

